
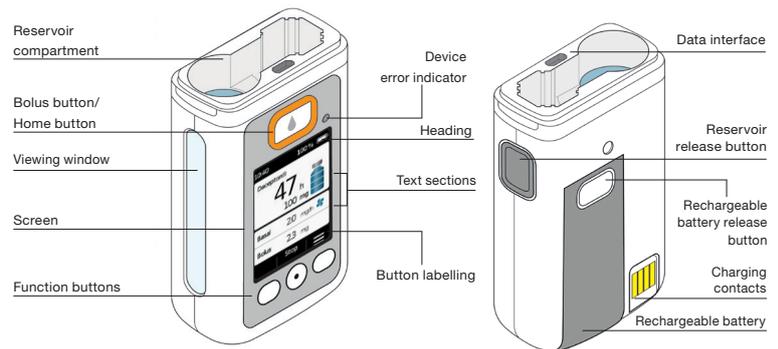


D-mine® Pump Quick User Guide

DESCRIPTION OF PARTS OF D-mine® PUMP THERAPY

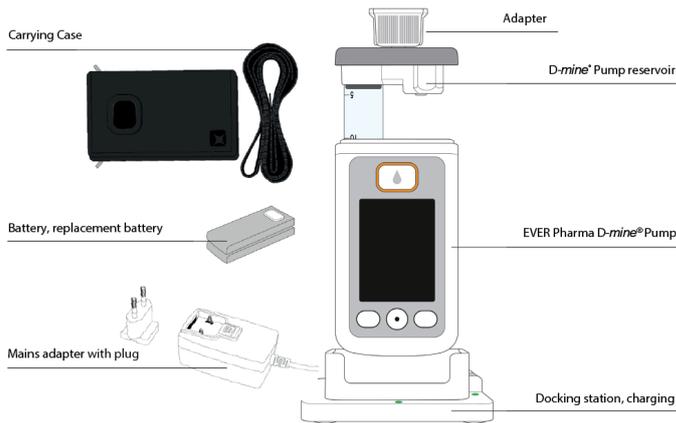





EQUIPMENT REQUIRED FOR D-mine® PUMP SET UP

- Ensure you have the following items:
 - D-mine® Pump
 - D-mine® Pump Docking Station
 - Dacepton® 20ml Vial
 - Infusion Line
 - D-mine® Pump Reservoir
 - Disinfection solution for skin and surfaces
- Ensure you have a clean work surface
- Wash and disinfect your hands thoroughly prior to setting up infusion
- Change the battery each day prior to setting up the infusion

i Dispose of sharps (such as used needles) as per local requirements (usually a sharps bin or equivalent)


PREPARING THE INFUSION

- Remove the reservoir from its packaging.
- Insert reservoir into pump clicking it firmly into place.
- Press „New Reservoir“ then confirm „The self-test is complete“
- Remove the orange cap from the vial and place it into the docking station.
- Turn the pump with the reservoir and tightened adapter upside down and let it vertically click into place on the vial in the docking station.
- Turn the pump upright with the vial attached and place it in the docking station. Check the placement of the adapter by turning it clockwise. Follow instructions on the screen to start the filling process. The pump must remain upright during the filling process.
- Grip the vial by the adapter and remove both from the reservoir by turning them counterclockwise. Dispose of these components with the household waste in accordance with Section 9.5 of your Instructions For Use.
- Check to ensure that the threaded connection on the reservoir is still clean. Remove a compatible infusion set from the packaging and connect it to the reservoir of your pump. Handtighten the threaded connection securely.

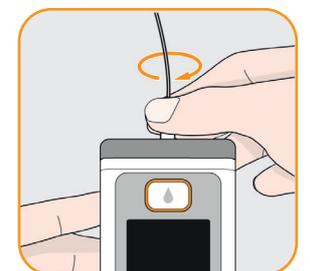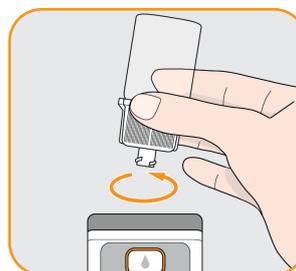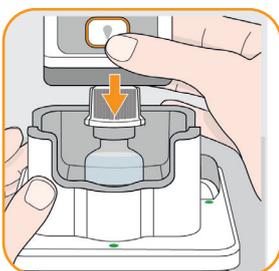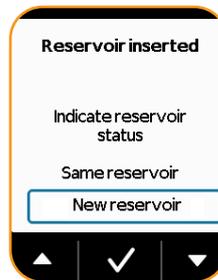

PRIMING AND STARTING THE INFUSION

9. Place the pump upright in the docking station and confirm .

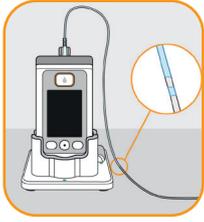


10. The pump display will show „Pump is being prepared“ - allow a moment for this. The pump has to remain upright during preparation.



11. The option for priming the infusion line will be shown - select either „Yes“ or „No“.

 Ask your specialist for advice on priming the infusion line

You may use the following injection sites:

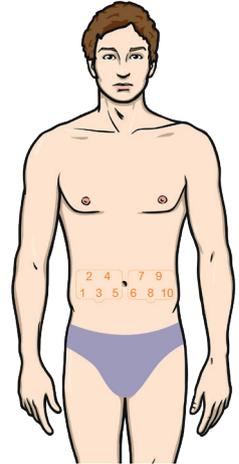

Ensure you rotate injection sites for each injection. Massage is helpful to reduce skin irritation

12. Put on infusion line as per advice of your specialist and confirm the step on the pump.



13. To start the infusion, press the „Start“ button for 3 seconds - the basal rate and bolus information will be shown on the display. The rotating propeller symbol indicates that the pump is running.




STOPPING THE DRUG DELIVERY

If the remaining drug will be used the following day, please leave the pump reservoir intact and close the reservoir sterily with a combination stopper overnight. You will need to remove the used infusion line and replace it when you are ready to begin the infusion again.

Refer to step 8 and connect a new infusion line to the reservoir of your D-mine® Pump. If the reservoir is empty, please continue with the steps below:

1. Stop the infusion by pressing „Stop“ on the pump
2. Remove the infusion line from the skin and the pump. Dispose of the infusion line safely.
3. Remove the used reservoir from the pump by firmly pressing the grey release button on the side of the pump.
4. Dispose of the used reservoir in a sharps bin and return the pump to the docking station for charging overnight.

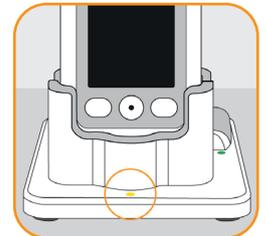

Visit us on

www.d-minecare.com

D-mine® Care