



FINAL TRANSPORTED PRESCRIBING POTULATION Designs of major Southon for inclusion. CRAIN TRANSPORTED PRESCRIBING POTULATION Designs of major Southern prescribed prices priced by the storm of method prices and the storm of major southern prescribed prices. Price 1 is 2 - 10. Our ceiting year of southern prices and the storm of major southern prices and the storm of the



### D-mine® Pump



The new D-mine® Pump for Dacepton® 5 mg/ml infusions in Parkinson's disease








# THE ONLY MG-BASED VOLUMETRIC PUMP SYSTEM.

## **Specifically developed for patients with Parkinson's Disease**

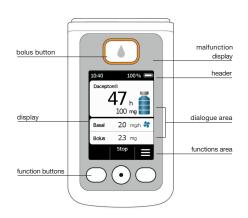

- mg-based for less dose errors
- automatic filling process for less handling steps and more safety
- individually programmable with 5 rates
- intuitive menu guidance in local language
- electronic filling control
- less time required for training
- data history

# INTUITIVE HANDLING MAKES LIFE EASY.

## Easy handling in just 5 steps for more self-autonomy



### **Step 1: Preparing**

Take the sterile D-mine® Pump reservoir out of the blister and connect it with the pump.

### **Step 2: Attaching**

Fix the entire pump with the reservoir and attached adapter onto the vial.



Put the pump in the docking station and start the filling process.



Step 4: Connecting
Connect the infusion
set to the pump.



### Step 5: Starting

After the priming process puncture the skin and start the infusion.

