

Handling Care Instructions for D-mine® Pump Infusion Therapy



D-mine® Pump

HANDLING CARE INSTRUCTIONS

For the treatment of Parkinson's disease patients with continuous subcutaneous infusions of Dacepton® (Apomorphine Hydrochloride 5 mg/ml in 20 ml vial).

These instructions are provided on the understanding that the user or patient was adequately trained by the hospital or other certified persons and knows the handling steps for his/her infusion therapy. It does not replace the information of the instructions for use of the pump system or the patient information leaflet of the pharmaceutical product.

Dispose of used and sharp materials according to local requirements.

1. PREPARATION

- Put the D-mine® Pump in place.
- Make sure you have a clean working space.
- In addition to disinfection solution, disinfection pad and patch you will need the following disposables:
 - Apomorphine 20 ml vial
 - Special D-mine® Pump Reservoir 20 ml for your infusion pump
 - Infusion set (catheter)



2. WORKING STEPS

1. Hands should be washed and disinfected with disinfection solution.



2. Take a new D-mine® Pump Reservoir and remove it from the packaging.



3. Insert the reservoir with the pre-assembled adapter into the opening of the pump.



You are now ready for the pump filling process.

4. Take the vial and remove the orange cap. Do not remove the grey plunger.



5. Place the vial upright into the indicated space of the docking station of the pump.



6. Attach the pump with the adapter to the vial vertically from above.



The pump automatically trans-

fers the drug from the vial into

minutes and you can follow the

the filling process. Please verify

and press the confirm button

the reservoir. It takes a few

progress on the screen. The pump indicates the end of

Please note:

7. With the vial attached turn the pump around and place it upright into the docking station.



9. Remove the vial with the at-

tached adapter from the pump

by turning anti-clockwise.


8. To start the filling process pump.



press the confirm button on the





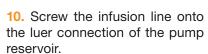



11. The needle is placed according to the recommendations of the Healthcare Professional.



To start the infusion, please refer to the instructions for use of the pump system. Under consultation with the Parkinson's disease specialist, the dose will be determined and set on the pump.

Our mission is to improve the quality of life of Parkinson's disease patients. Each and every day - step by step.

www.d-minecare.com



EVER Neuro Pharma GmbH Oberburgau 3, 4866 Unterach/Austria www.everpharma.com